REF Catalogue number 751720
Device

IVD In Vitro Diagnostic Medical

Alternate sample diluent vial for use with ImmunoCard STAT! HpSA

INTENDED USE

Simple Sample is an alternate sampling unit that can be used in place of the Sample Diluent vial provided with ImmunoCard STAT! HpSA.

SUMMARY AND EXPLANATION OF THE TEST

Specimens to be tested with ImmunoCard STAT! HpSA must first be sampled and diluted in Sample Diluent. When provided as part of the ImmunoCard STAT! HpSA test kit, Sample Diluent is supplied in individual plastic vials with applicator sticks attached to caps. The applicator stick is used to transfer a small portion of the stool sample to the Sample Diluent vial. The vial is capped and the contents mixed. Once the sample is diluted it is ready to be tested.

Simple Sample contains the same liquid Sample Diluent as found in the ImmunoCard STAT! HpSA test kit. Sample Diluent is provided in a plastic tube with a sampling brush, rather than an applicator stick. The sampling brush is used to transfer a small portion of the stool sample to the Sample Diluent. The Simple Sample is then closed and contents mixed. Sample is now ready to be tested.

PRINCIPLE OF THE TEST

Simple Sample is an alternate Sample Diluent vial made of two units: (1) a small brush attached to a plastic handle, and (2) a sealed, flexible plastic tube filled with Sample Diluent. One end of the Sample Diluent tube is sealed with removable foil and the other has a dropper tip with a screw cap. The foil seal is removed from the Sample Diluent tube and the sampling brush is used to collect a small portion of stool from a sample. The stool coated sampling brush is inserted into the Sample Diluent tube. The two units are closed, and the sample is mixed. When ready to test, the screw cap is removed from the dropper tip end of the Sample Diluent tube and the tube is squeezed to dispense diluted sample onto the test device.

REAGENTS

Each Simple Sample contains two units packaged in a plastic bag. The brush unit consists of a small brush attached to a plastic handle. The Sample Diluent unit consists of a flexible plastic tube containing Sample Diluent, a buffered salt solution containing sodium azide (0.095%) as a preservative. The Sample Diluent tube unit is sealed with foil. Remove the foil seal before use. Store at 2-25 C when not in use. Do not freeze.

MATERIALS PROVIDED PER SIMPLE SAMPLE BAG

1. Sample Diluent in a sealed tube with dispensing tip. Use as supplied
2. Stool sampling brush.
3. Stool collection paper

The maximum number of tests obtained from this test kit is listed on the outer box.

The Simple Sample system is diagrammed below:

MATERIALS NOT PROVIDED

1. ImmunoCard STAT! HpSA Test Kit (Product code 750720) used for the detection of *H. pylori* stool antigens in human samples (Kit sold separately.)
2. Disposable latex gloves (Stool samples are considered potentially hazardous material.)
3. Pen or marker
4. Stool collection containers without transport media such as cups or tubes (optional)

PRECAUTIONS

1. This reagent can only be used with ImmunoCard STAT! HpSA.
2. Do not spill the contents of the Sample Diluent tube during sample preparation. The liquid volume is critical for proper testing of the stool sample.
3. If the liquid is spilled from the tube, do not proceed with stool sampling. Discard and obtain a new Simple Sample system.
4. Do not use beyond the labeled expiration date.
5. Do not use Simple Sample tube if the foil seal is broken or torn before use. Incorrect test results may occur if the seal has been damaged.
6. Do not use Simple Sample if Sample Diluent in tube is turbid. Turbidity may be a sign of microbial contamination.
7. The collection brush is designed to collect an aliquot from an already collected stool specimen. The collection brush is not designed to collect stool directly from the patient. The collection brush is not designed to be used as a rectal swab.
8. This device is not designed for use with control reagents or survey samples.

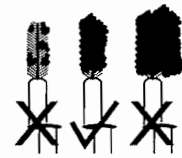
SAMPLE COLLECTION AND PREPARATION

This test should be performed by qualified personnel per local regulatory requirements.

Acceptable specimen types: Simple Sample may be used with liquid, semi-solid, or solid stools. Watery stools composed mainly of fluid with little or no solid matter are not appropriate for testing.

Specimen Collection: The stool specimen should be collected on the collection paper (provided with the Simple Sample kit), or in an alternate collection cup. Stool in transport media, on swabs, or mixed with preservatives is not appropriate for testing.

1. To collect a stool sample, remove the collection paper from the plastic bag. Follow the collection instructions printed on the stool collection paper. (Alternatively, sample can be collected into a collection cup.)
2. Unroll the label from the sampling brush. Fill out the patient information on the label.
3. Carefully remove the aluminum seal at the top of the Sample Diluent tube. Hold the tube upright so as not to spill the liquid contained in the tube.
4. Use the sampling brush to collect an appropriate amount of stool sample as shown in the pictures below. Rotate the brush in the stool to ensure complete coverage of the sampling brush.


5. Insert the coated brush into the tube and tighten the two units together until securely closed.
6. Mix the sample by inverting the tube several times.
7. The diluted sample can be stored at room temperature (17-25 C) for no more than 48 hours before testing. The diluted sample can be stored refrigerated (2-8C) for no more than 72 hours before testing.
8. Complete testing in Laboratory Area as described in the ImmunoCard STAT! HpSA test procedure for Simple Sample.

LIMITATIONS OF THE PROCEDURE

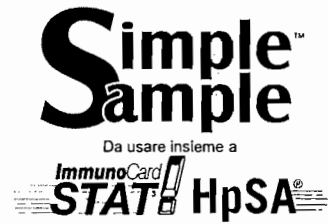1. Simple Sample is to be used only with ImmunoCard STAT! HpSA (Product code 750720). See the complete ImmunoCard STAT! HpSA package insert to obtain information regarding specific performance characteristics of the test.
2. Failure to collect sufficient stool with the Simple Sample brush may result in a falsely negative test result with the ImmunoCard STAT! HpSA test. Conversely, addition of too much stool may result in an invalid test result due to the inhibition of proper sample flow.
3. Neither Simple Sample nor ImmunoCard STAT! HpSA are designed for use with watery, diarrheal stools. Do not use these sample types with these products.

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