

# SARS-CoV-2 Ag Rapid Test (Saliva test)

## PROCEDURE CARD



### DIRECTIONS FOR USE

#### Specimen Collection

The oral fluid specimen should be collected using the collection device provided with the kit. Follow the detailed Directions for Use below. No other collection devices should be used with this assay. Oral fluid collected at any time of the day may be used.

Allow the test, extracted specimen and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

1. Remove the test cassette from the sealed foil pouch and use it within one hour. Best results will be obtained if the test is performed immediately after opening the foil pouch.
2. Invert the specimen collection tube and add 3 drops of the extracted specimen (approx. 75µl) to the specimen well(S) and then start the timer.
3. Wait for the colored line(s) to appear. Read the result at 15 minutes. Do not interpret the result after 20 minutes.

