




ImmunoCard **STAT! NOROVIRUS**


Rapid test for the detection of Norovirus in stool specimens

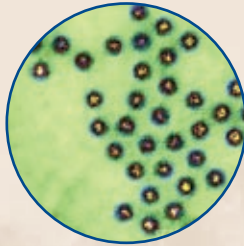


Be ahead of the outbreaks with routine testing

- Rapid detection of the most common cause of gastroenteritis in people of all ages
- Prompt diagnosis of potential outbreaks
- Results can be useful in outbreak containment and infection control
- One step test provides results in 15 minutes
- Simultaneous detection of Genogroups I and II
- Room temperature storage


Test Procedure

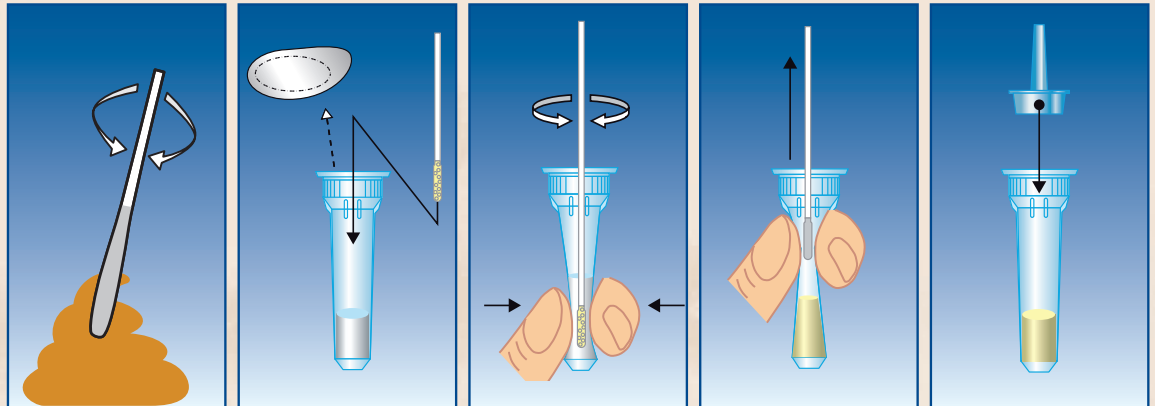

1. Collect the stool specimen with the swab supplied with the kit.
2. Remove the aluminium seal from the Sample Diluent Tube. Immediately place the swab into the Sample Diluent Tube.
3. Vigorously mix the solution by rotating the swab against the side of the Sample Diluent Tube.
4. Release as much liquid as possible from the swab by squeezing the sides of the tube as the swab is withdrawn. Discard the swab.
5. Firmly attach the Dropper Cap with Filter to the Sample Diluent Tube.

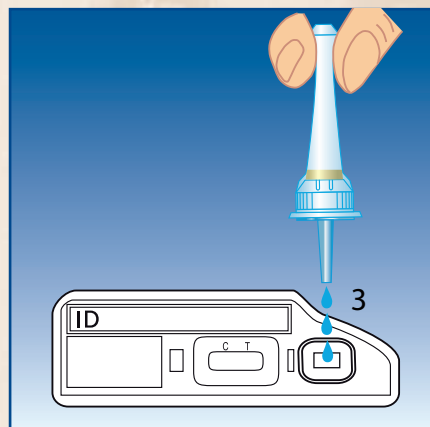

6. Turn the tube upside down and dispense 3 drops of diluted specimen to Sample port of the Test Card. Incubate 15 minutes at 15-30°C.

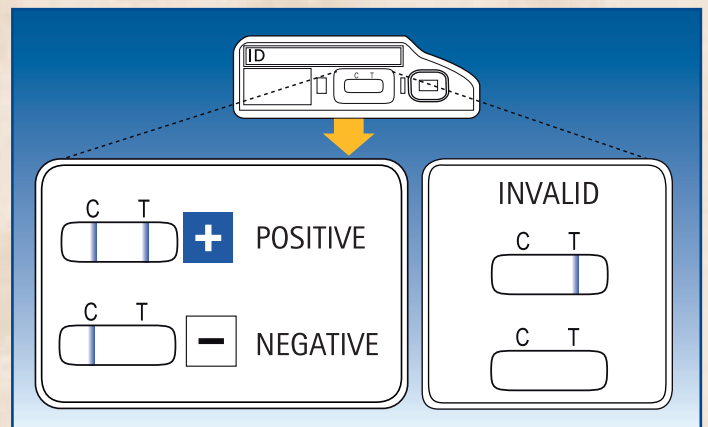

7. Read Control (C) and Test (T) zones for the presence or absence of a blue line.

Ordering Information

Products	Test Size	Catalog #
<i>ImmunoCard STAT!® NOROVIRUS</i>	20 tests	757020
<i>ImmunoCard STAT!® NOROVIRUS Positive Control</i>	1 test	757001
<i>Rapid Strip ROTA/ADENO</i>	20 tests	751120
<i>ImmunoCard STAT!® ROTAVIRUS</i>	30 tests	750030
<i>ImmunoCard STAT!® ADENOVIRUS</i>	30 tests	751030
<i>Premier™ Rotaclone®</i>	48 Microwells	696004
<i>Premier™ Adenoclone®</i>	48 Microwells	696007
<i>Premier™ Adenoclone® - Type40/41</i>	48 Microwells	696006



Meridian Bioscience Europe Srl
Via dell' Industria, 7
I-20020 Villa Cortese, Milano - Italy
Tel: +39 (0331) 43 36 36
Fax: +39 (0331) 43 36 16
E-mail: info@mdeur.com

Meridian Bioscience Europe sa/nv
Rue de l'Industrie, 7
B-1400 Nivelles - Belgium
Tel: +32 (67) 89 59 56
Fax: +32 (67) 64 41 92
E-mail: info@mdeur.be

Meridian Bioscience Europe France
Le Quadra Promenade des Anglais, 455
F-06299 Nice Cedex 3 - France
Tel: +33 (4) 93 18 72 10
Fax: +33 (4) 93 18 72 11
E-mail: info@meridianbioscience.fr

Meridian Bioscience Europe bv
Halderheiweg, 6
NL-5282 SN Boxtel
The Netherlands
Tel: +31 (411) 62 11 66
Fax: +31 (411) 62 48 41
E-mail: meridian.info@planet.nl

Meridian Bioscience, Inc.
Corporate Office
3471 River Hills Drive
Cincinnati, Ohio 45244, USA
Tel: +1 (513) 271 37 00
Fax: +1 (513) 271 37 62
E-mail: mbi@meridianbioscience.com

DISTRIBUTED BY

www.mdeur.com
www.meridianbioscience.com