




ImmunoCard[®] MYCOPLASMA

Rapid EIA for the detection of IgM to *Mycoplasma pneumoniae*

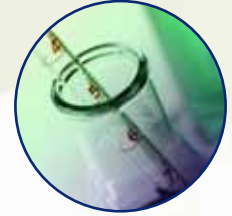

The only rapid test that detects IgM antibody exclusively – for the diagnosis of early/acute illness rather than convalescent disease.

- “ The Meridian ImmunoCard[®] is the test of choice, especially for children, if only acute phase sera are available for testing. ¹ ”
- “ Immunocard[®] Mycoplasma IgM assay is a valid and simple procedure which can reduce technologist time and turnaround time for labs analyzing small numbers of specimens. ² ”


- Four simple steps to process
- No calculations necessary
- Rapid results in 9 minutes
- Sensitivity: 88% Specificity: 90%³

¹Talkington, D, Thaker, WL. *Analysis of Complement Fixation and Commercial Enzyme Immunoassays for Detection of Antibodies to Mycoplasma pneumoniae in Human Serum.* Clinical and Diagnostic Laboratory Immunology, Sept 2000; 778-780.


²Alexander, et al. "Performance of Meridian ImmunoCard[®] Mycoplasma Test in a Multicenter Clinical Trial." *Journal of Clinical Microbiology*, May 1996; 1180-1183.

³ImmunoCard[®] Mycoplasma Package Insert.

ImmunoCard® Mycoplasma Test Procedure

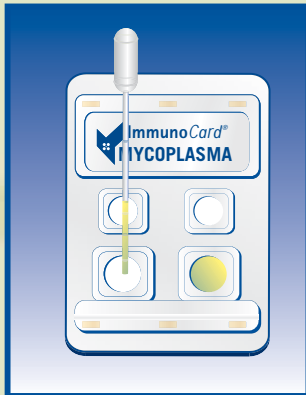

1. Add 2 drops of serum to both lower ports. Incubate for 2 minutes at 22-25C.

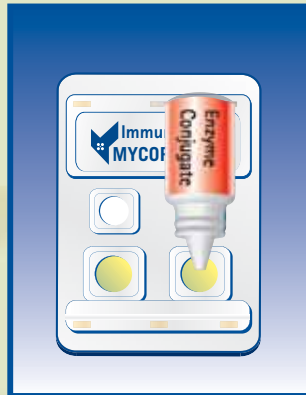

2. Add 3 drops of Conjugate to both lower ports. Incubate for 2 minutes at 22-25C.






3. Add 3 drops of Wash Buffer to both upper ports.



4. Add 2 drops of Substrate to both upper ports. Incubate for 5 minutes at 22-25C. Interpret visually.

Interpretation of Results

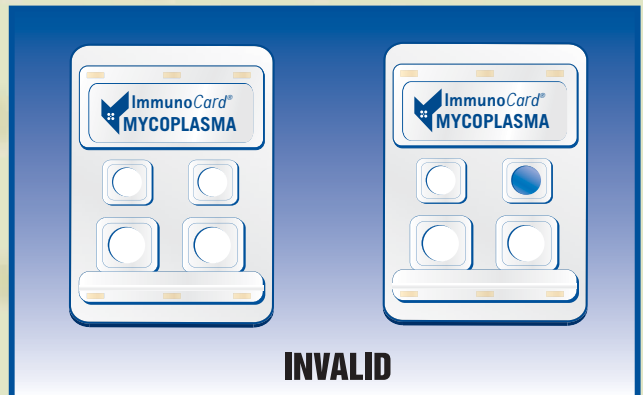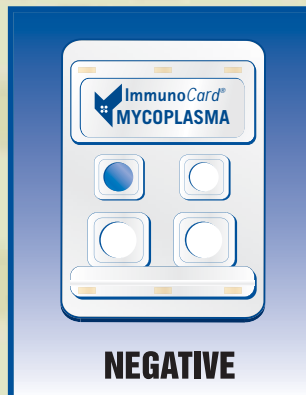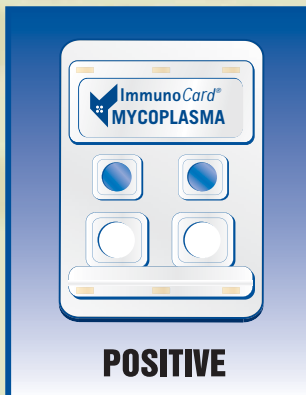

*This chart does not contain complete instructions for use. For further information, please refer to the package insert.

Ordering Information

ImmunoCard® Mycoplasma

Test Size
30 Tests

Catalog #
709030

Other Meridian Products

ImmunoCard STAT!® Rotavirus

Test Size
30 Tests

Catalog #
750030

ImmunoCard STAT!® Strep A

30 Tests

750230

ImmunoCard STAT!® Mono

30 Tests

750330



USA/Corporate Office:
3471 River Hills Drive
Cincinnati, Ohio 45244
Telephone: 513.271.3700
Orders/Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
www.meridianbioscience.com



Belgium / France / Italy / The Netherlands

©2006 ICMYC001