



TRU FLU®

Rapid test for the detection of Influenza A & B

Gain time, space, and peace of mind

Limit pathogen exposure in the laboratory

- Confidently follow the CDC guidelines for influenza testing at Biosafety Level 2, which recommend the use of biological safety cabinets *
- Take advantage of the closed test design of TRU FLU®, which is ideal for testing under the hood
- TRU FLU® is closed immediately after patient sample addition, and remains closed for interpretation and disposal

Optimize your lab's workflow

- The convenient work station supplied with TRU FLU® allows six tests to be performed in the space of one traditional card test
- TRU FLU® tests are easily batched
- Save time with easy, ready to use external controls

Limit susceptibility to variation in operator technique


- Easy, single step sample addition
- Test for both Flu A and Flu B with the same sample application
- Vertical test strip can help reduce flow issues which occur with traditional flat card tests

TRU FLU® is unique among rapid influenza test devices


- Test under the hood in compliance with CDC guidelines without concern about airflow
- Easy to use controls are validated with both TRU FLU® and TRU RSV®

* U.S. Department of Health and Human Services, Centers for Disease Control and Prevention and National Institutes of Health. Biosafety in Microbiological and Biomedical Laboratories. Fifth Edition 2007. U. S. Government Printing Office, Washington, DC: 2007

TRU FLU® Test Procedure

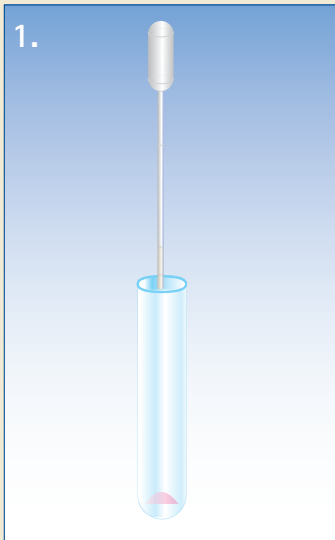

1. Add 100 µL of Sample Diluent (second mark) to the conjugate tube. Mix well.

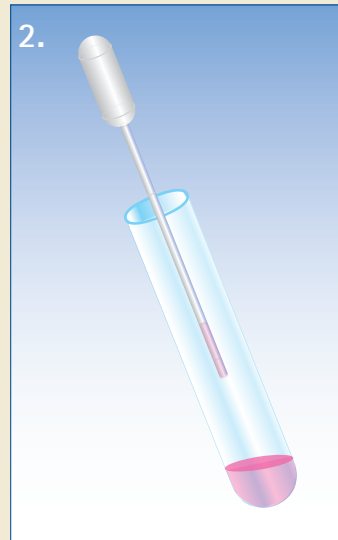


2. Draw Sample to second mark (100 µL) on transfer pipette, add to tube. Mix by squeezing pipette bulb 3 times or vortex for 10 seconds.



3. Add Test Strip to tube and secure in place. Incubate at 20–25 C for 15 minutes. Read results within 1 minute.

RESULTS

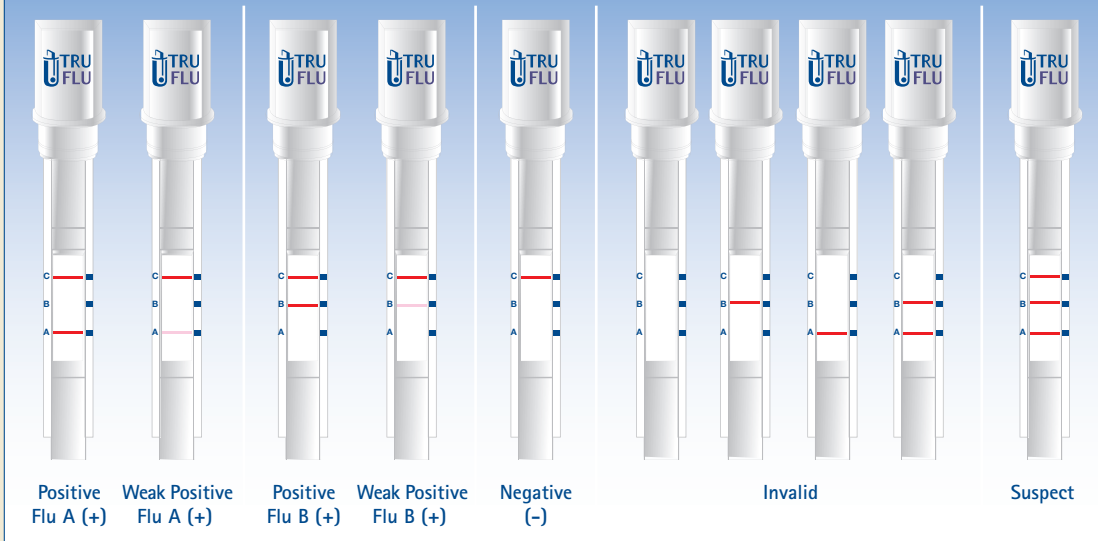

*This chart does not contain complete instructions for use. For further information, please refer to the package insert.

Ordering Information	Test Size	Meridian#	Cardinal#	Fisher#
TRU FLU®	32 Tests	751230	TBD	TBD
For rapid Respiratory Syncytial Virus testing	32 Tests	751330	TBD	TBD
TRU RSV®	3 vials of 8 tests each	751110	B1043-37A	23030825
FLU/RSV positive control				
TRU FLU® & TRU RSV®				

USA/Corporate Office
3471 River Hills Drive
Cincinnati, Ohio 45244
Telephone: 513-271-3700
Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
www.meridianbioscience.com

Europe b.v.
Halderheiweg, 6
5282 SN Boxtel
The Netherlands
Tel: +31 (411) 62 11 66
Fax: +31 (411) 62 48 41
www.mdeur.com

Europe s.a./n.v.
Rue de l'Industrie, 7
1400 Nivelles - Belgium
Tel: +32 (67) 89 59 59
Fax: +32 (67) 89 59 58
www.mdeur.com

Europe France
Le Quadra
455, Promenade des Anglais
06299 Nice Cedex 3 - France
Tel: +33 (4) 93 18 72 10
Fax: +33 (4) 93 18 72 11
www.mdeur.com

Europe S.r.l.
Via dell'Industria, 7
20020 Villa Cortese (MI) - Italy
Tel: +39 (0331) 43 36 36
Fax: +39 (0331) 43 36 16
www.mdeur.com